

Complete Kit



Zeofresh[™]
AUSTRALIA


BIOHAZARD SPILL KIT

A WELL DESIGNED SPILL KIT IS HIGHLY RECOMMENDED

- All blood and body fluid spills should be dealt with immediately.
- The use of protective clothing worn over a uniform or work clothing (e.g. apron, gloves, face mask etc.) will avoid undue contamination when exposed to blood or body fluids.
- The ZeoFresh Biohazard Spill Kit is designed with the staff in mind and will minimise the risk of potential infection that may occur in the workplace.

MULTIPLE USES

The ZeoFresh Biohazard Spill Kit is useful for medical practices, dental practices, clinics, ambulance services, first-aid rooms, hospital wards, and nursing homes. Also appropriate for schools, sports facilities, home health care settings and the general work place.

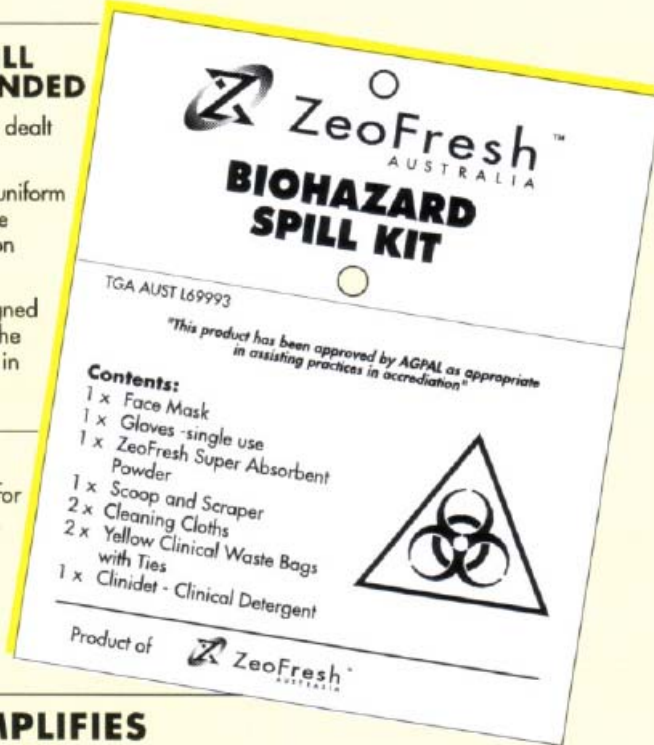

SAVES TIME & SIMPLIFIES THE PROCEDURE

When an emergency spill occurs, kits save time because there is no frantic search for a disposable gown, gloves & clinical detergent etc. It is all available in the one handy kit.

EASY TO HANDLE

The ZeoFresh super absorbent powder solidifies the blood/body fluid and renders a mass that is safe and easy to handle.

REFILL KITS AVAILABLE

Maintain a backup refill kit or order immediately after use.

"All Health care establishments should have a management system in place for dealing with blood/body fluid spills. Protocols for spills management should be included in Procedure Manuals and highlighted in ongoing education and training programs. A Biohazard Spill Kit permits prompt, safehandling, removal and disposal of potentially contaminated blood/body fluid spills."

"This product has been approved by AGPAL as appropriate in assisting practices in accreditation"

Available from:

Easy to use



MAJAC MEDICAL PRODUCTS

"Let's Get It Right"

Sales Inquiries Tel:1300 138 578 Fax:1300 138 612
Business Office Tel: +61 7 3265 6355 Fax +61 7 3265 2729
e-Mail: sales@majacmedical.com.au
www.majacmedical.com.au

Your Complete Range of Cleaning & Infection Control Products

INSTRUCTIONS FOR USE:

1

Empty contents onto a clean surface. Don the protective clothing. Apron, face mask, gloves (*and eye wear where risk of splashing is possible)

(Fig 1)



(Fig 1)

2

Open Gusseted clinical waste bag (yellow) by placing hands inside bag & pressing firmly on base. Place in an upright, free standing position. (Fig 2)



(Fig 2)

3

Open ZeoFresh absorbent powder satchel and apply evenly over spill area. Allow 1-2 minutes for powder to absorb spill and ordours. (Fig 3)

(Fig 3)




Using the scraper and scoop pick up the substance and place into the *flat standard bio-waste* (yellow) bag. Then place scraper and scoop into the bag. Now seal the bag securely with one of the ties provided and place into the upright, gusseted clinical waste (yellow) bag.

Avoid cross contamination. Do not touch the outside of the gusseted bag.

(Fig 4)

4

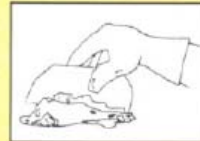

6

Remove apron, face mask and gloves and also place in gusseted clinical waste (yellow) bag. Seal the bag securely with second tie then *dispose of bag in accordance with Government Regulations.*

(Fig 6)



(Fig 7)

7

Wash and dry hands..... Thoroughly. (Fig 7)

5

Clean the area with the Clinical Detergent Solution (Clinidet) using one of the enclosed cleaning cloths. Contain the spill by wiping from the outside towards the centre. Place the contaminated cloth into the gusseted clinical waste (yellow) bag. Dry the area with the second cloth, then also dispose of by placing in the gusseted clinical waste (yellow) bag.

Avoid cross contamination. Do not touch the outside of the gusseted bag.

Protect yourself and others from potentially contaminated spillage. Dispose of spillages using the

ZeoFresh
**BIOHAZARD
SPILL KIT**

* Note. Eye wear must be cleaned and disinfected before reuse or disposed of along with contaminated waste.



MAJAC MEDICAL PRODUCTS

"Let's Get It Right"

Sales Inquiries Tel:1300 138 578 Fax:1300 138 612
Business Office Tel: +61 7 3265 6355 Fax +61 7 3265 2729
e-Mail: sales@majacmedical.com.au
www.majacmedical.com.au

Your Complete Range of Cleaning & Infection Control Products